
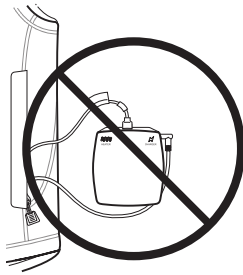


Instructions for Use

Consult your hospital's policies regarding use of the Solor vest in the OR.

Operation

1. Charge battery for at least 3 hours prior to first use.
2. Insert the heater plug into heater port on battery.

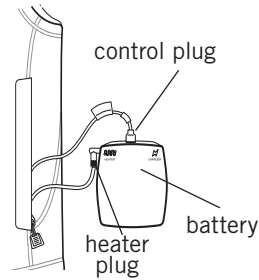


Do not insert heater plug into charger port

3. Insert the control plug into port located at top of battery.
4. Insert battery into inner left front pocket.

5. Hold power button for 2 seconds to switch heating system from:

- High (solid light)
- Low (flashing light)
- Off (no light)

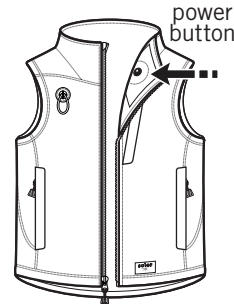


Charging the Battery

Before using the Solor vest for the first time, charge the battery for a minimum of three hours. To reach optimum battery operation, 2-3 complete charging cycles (i.e., full charge/use/battery depletion) may be required.

NOTE: To avoid damaging or dropping the battery, take off the vest before removing battery.

1. Remove battery from vest connections. Do not remove battery from vest in OR.
2. Insert charger plug into charger port on battery. Do not charge battery in OR.

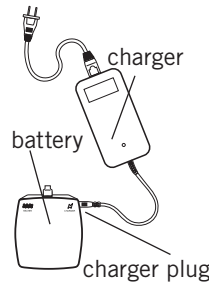

Do not charge battery while connected to vest



The red light on the front of the charger will light up when checking the level of battery power. A fully depleted battery pack will take approximately 3 hours to fully recharge.

The red light stays on continuously while the battery is being re-charged. When the battery is completely charged, the red light turns to green.

NOTE: The charger supplied has been chosen specifically for the battery pack supplied. **DO NOT USE ANY OTHER CHARGER.**

Cleaning Instructions

1. Remove battery from vest connections.
2. Secure cables in pocket and zip pocket closed.
3. Machine wash cold. Hand washing is recommended for spot cleaning.
4. Hang dry only.

Things to Avoid:

- Do not bleach
- Do not dry clean
- Do not iron
- Do not tumble dry
- Do not get battery wet.

Storage

Store battery packs in room temperature and between 30-50% humidity.

Troubleshooting

The heating system has been designed for safe and reliable use. Do not replace or interfere with the electrical components in the heated jacket or the battery and charger unit. Contact Arizant Customer Service.

What happens if my heated vest is torn?

1. Do not attempt to replace or interfere with the electrical components in the heated vest. Contact Arizant Customer Service.

NOTE: Before using the Solor vest for the first time, charge the battery for a minimum of three hours. To reach optimum battery operation, 2-3 complete charging cycles (i.e., full charge/use/battery depletion) may be required.

What happens if my heated vest doesn't get warm or the battery prematurely loses its charge?

1. Remove battery from vest connections.
2. Recharge the battery for three hours, even if the charger light is green.
3. Reconnect the battery to the vest, confirming that the control plug and heater plug are properly fitted to the correct ports.
4. Hold power button for 2 seconds.
5. If the power button does not illuminate or the vest does not feel warm, contact Arizant Customer Service.
6. If the power button illuminates, use the vest. A fully charged battery should hold its charge between 5 and 6 hours on the low setting and 3 hours on the high setting. If the battery still prematurely loses its charge, contact Arizant Healthcare Customer Service.

Solor™ Warming Vest Limited Warranties

Thank you for purchasing the Solor™ vest, which consists of a garment with an integral heating element and an integrated operating switch, a battery, and a charger and power cord (the "Product") from **Arizant Healthcare Inc.** ("Arizant Healthcare").

For your own records, staple or attach your sales receipt to this warranty (**SAVE THIS WARRANTY INFORMATION**).

Warranty

Arizant Healthcare warrants to the original end user ("User") of this battery-operated, warming vest that: the vest composed of the garment, the integral heating element and integrated operating switch, (the "Vest") will be free from defect in material and workmanship, under normal use, for a period of six (6) months from the date of delivery to User; the individual lithium-ion battery that is sold with the Vest and any additional batteries the User purchases from **Arizant Healthcare** (the "Batteries") will be free from defect in material and workmanship, under normal use, for a period of six (6) months from the date of delivery to User; and the charger and power cord that are sold with the Vest will be free from defect in material and workmanship, under normal use, for a period of six (6) months from the date of delivery to User. The warranties do not apply to Product defects resulting from damage, misuse, abuse or alteration. Unless otherwise determined by law,

ARIZANT HEALTHCARE SHALL NOT BE LIABLE FOR ANY PERSONAL INJURY, PROPERTY DAMAGE OR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY KIND RESULTING FROM MALFUNCTIONS, DEFECTS, MISUSE, IMPROPER INSTALLATION OR ALTERATION OF THIS PRODUCT, OR USER FAILURE TO FOLLOW THE INSTRUCTIONS FOR USE.

This warranty does not affect Users' mandatory rights, if any, under applicable law.

Distributors are not Users unless they have purchased a Unit for their sales representatives' use.

This limited warranty applies only to User and is non-transferable.

Arizant Healthcare will also provide the User with a full refund or exchange if the User returns the Product within ten (10) business days from the date of delivery to User. The Vest must not have been worn, used, or laundered, and the original tags must remain attached. During this period, the Vest may have been fitted for sizing and comfort and may have been powered on. **IF TAGS ARE DETACHED, RETURNS WILL NOT BE ACCEPTED AND CREDIT WILL NOT BE ISSUED.**

These warranties are the only warranties extended by **Arizant Healthcare** to

